ENGLISH

Instructions for use

Intended Use:

Provox Skin Barrier is a single use wipe for laryngectomized patients that forms a barrier between Provox Adhesive and the skin.

CONTRAINDICATIONS:

Hypersensitive or breached skin or known allergies to ingredients.

Application:

Clean the skin around the stoma and let dry. Apply the Provox Skin Barrier to the skin around the stoma that will be covered by the adhesive. Allow the barrier to dry. Apply a Provox Adhesive.

PRECAUTIONS

- Avoid deep inhalation of solvent during application and be careful that liquid does not drip into stoma.
- In case you experience signs of irritation or redness, discontinue use of product. If the irritation persists, consult your physician.

WARNINGS

- For external use only.
- Keep out of reach of children;
- Avoid contact with eyes. In the case of accidental contact, flush eyes well with water;
- Do not apply directly to open wounds and mucous membranes;
- Single patient use only. Re-use may cause cross contamination.
- Do not cut the tissue in pieces, small pieces may be aspirated.

Disposal:

Used product may be a potential biohazard. Make sure that you follow local requirements for disposal.

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© Atos Medical AB, 2016.
Atos Medical AB