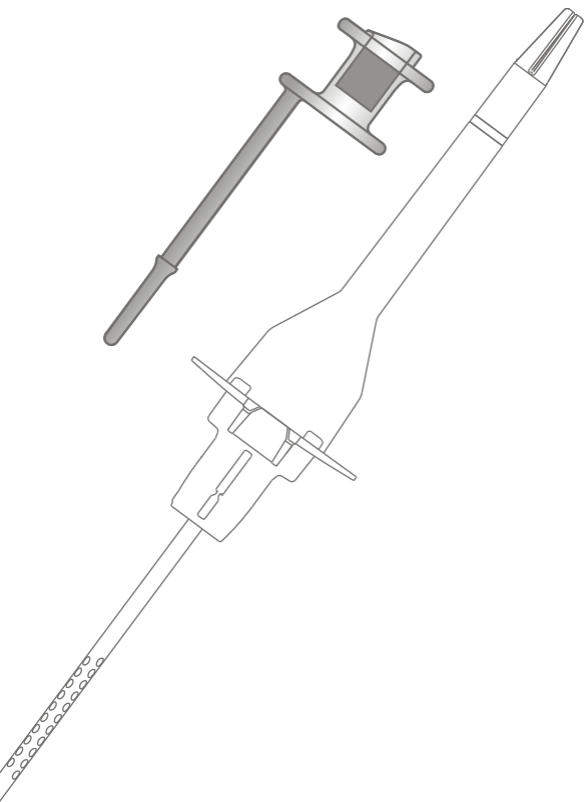
PROVOX[®] Vega[™]

Voice Prosthesis with SmartInserter[™]

Instructions for Use
Clinician's

**Rx
ONLY**

CE 0413

STERILE EO

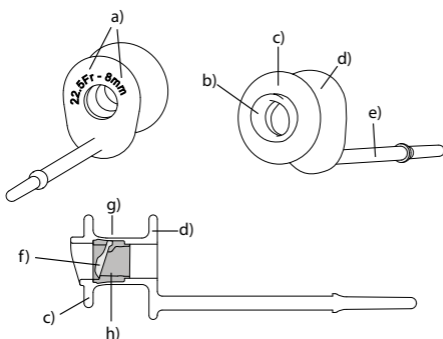
Atos

Atos Medical Your voice

Illustrations


1) Provox® Vega™ voice prosthesis

- a) Size information (shaft diameter and length between flanges)
- b) Prosthesis Hood
- c) Esophageal Flange
- d) Tracheal Flange
- e) Safety Strap
- f) Valve Flap
- g) Prosthesis Shaft
- h) Radio-opaque fluoroplastic Valve Seat (resisting candida growth)









2) SmartInserter™ System






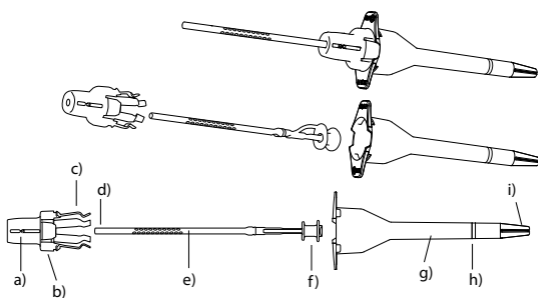
- a) Guide
- b) Connection Side
- c) Folding Device
- d) Distal tip
- e) Inserter Pin
- f) Voice prosthesis
- g) Loading Tube
- h) Step marker
- i) Tip

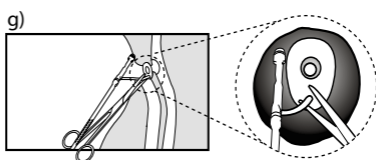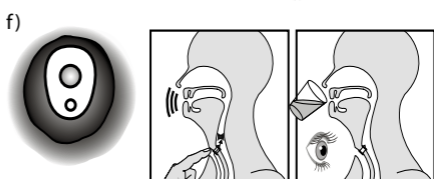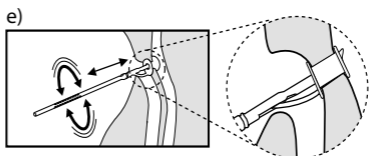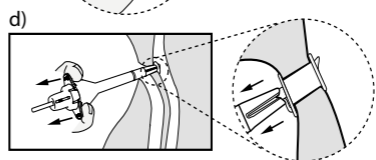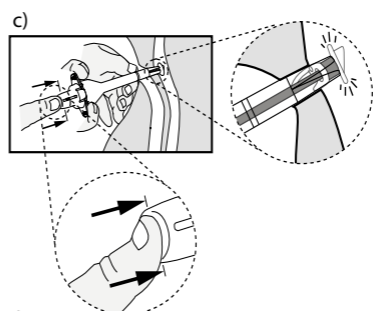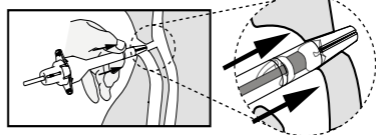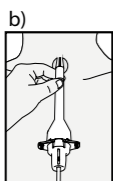
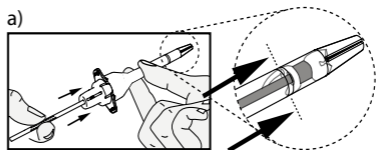
3) Insertion

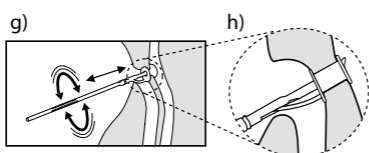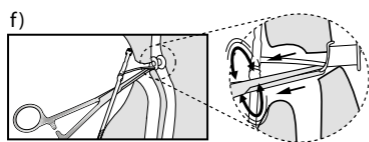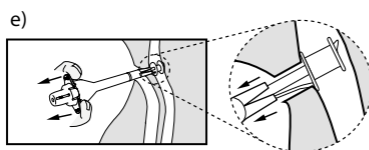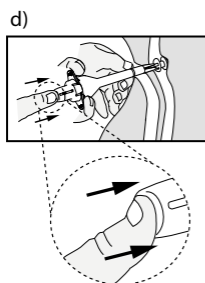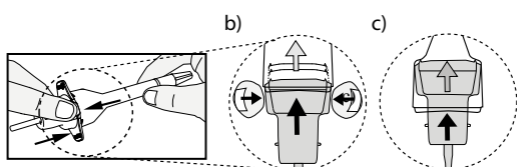
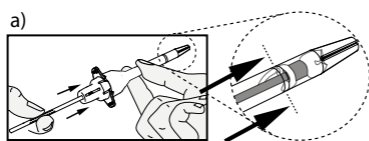
4) Over-shooting

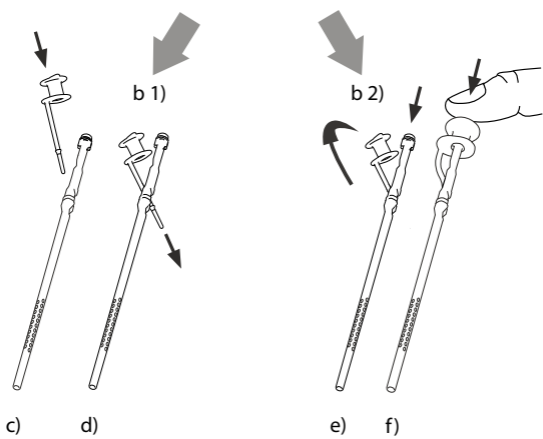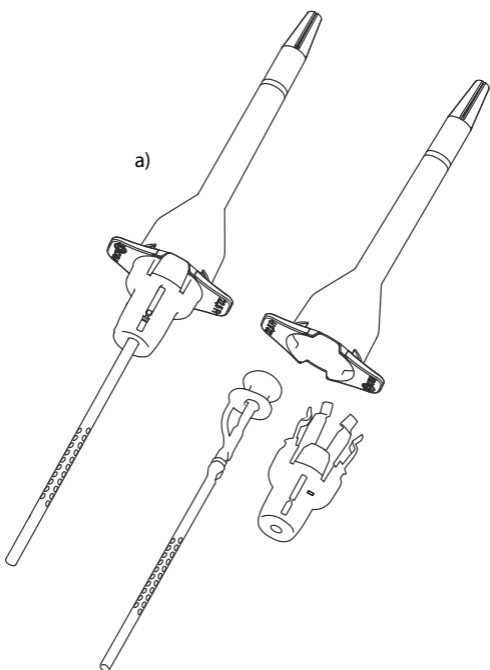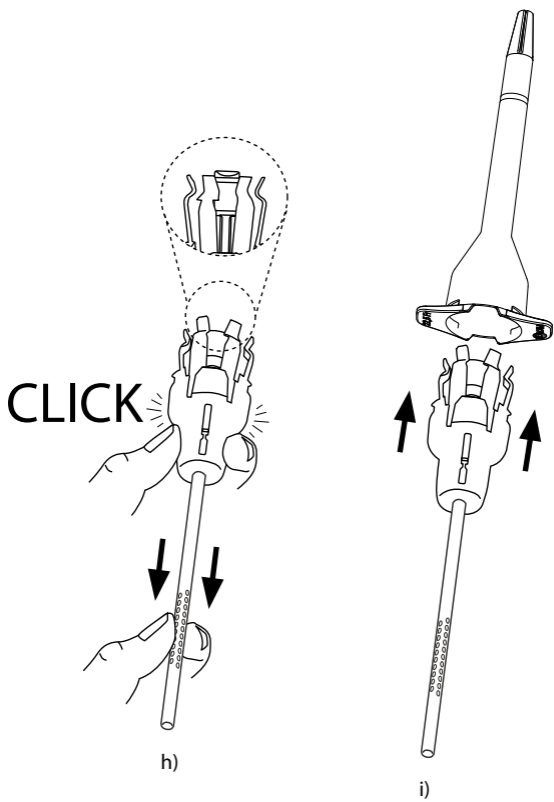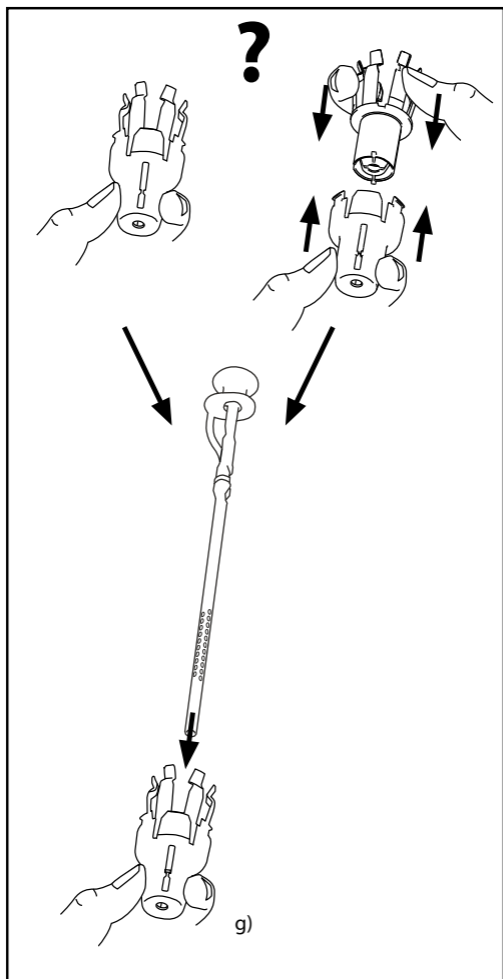5) Re-loading

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and Trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. Provox® Vega™ is a trademark of Atos Medical AB. For information about patents, please refer to the webpage www.atosmedical.com.

Contents

ENGLISH	9
DEUTSCH	13
NEDERLANDS.....	18
FRANÇAIS	24
ITALIANO.....	29
ESPAÑOL	34
PORTUGUÊS.....	39
SVENSKA	44
DANSK.....	49
NORSK.....	54
SUOMI	59
ČESKY	64
MAGYAR	69
POLSKI.....	74
HRVATSKI	79
TÜRKÇE	84
РУССКИЙ	89
日本語	95
한국어	100
繁體中文	105
עברית	115
العربية	121
ORDERING INFORMATION	122

Provox® Vega™

The Instructions for Use, which accompanies this product, may be revised from time to time and must therefore be reviewed prior to each procedure in which the product is used.

Contents

1. Descriptive information.....	9
1.1 Indications for use.....	9
1.2 Description of the device	9
1.3 CONTRAINDICATIONS	9
1.4 WARNINGS	10
1.5 PRECAUTIONS.....	10
2. Instructions for use.....	10
2.1 Preparation	10
2.2 Anterograde replacement procedure.....	10
2.3 Anterograde replacement procedure using voluntary overshooting	11
2.4 Reloading the rovox SmartInserter.....	11
3. Important patient information	12
3.1 General information	12
3.2 Prosthesis maintenance	12
4. Adverse events/Trouble shooting information	12
5. Additional information	12
5.1 Compatibility with MRI, X-ray and radiation therapy.....	12
5.2 Device lifetime.....	12
5.3 Compatibility with antifungal medicine.....	13
5.4 User assistance information	13

1. Descriptive information

1.1 Indications for use

The Provox Vega Voice Prosthesis is a sterile single use indwelling voice prosthesis intended for voice rehabilitation after surgical removal of the larynx (laryngectomy). Cleaning of the voice prosthesis is performed by the Patient while it remains in situ.

The Provox SmartInserter is a sterile single use device intended for anterograde replacement of the Provox Vega Voice Prosthesis. This replacement procedure is carried out by a medical doctor or a trained medical professional in accordance with local or national guidelines.

The Provox SmartInserter is not intended to be used for insertion of a voice prosthesis in a freshly made puncture.

1.2 Description of the device

General

The Provox Vega is a one-way valve (prosthesis) that keeps a TE-puncture open for speech, while reducing the risk of fluids and food entering the trachea. The Provox Vega voice prosthesis is not a permanent implant, and needs periodic replacement. The prosthesis (figure 1) is available in different diameters and several lengths. The device is made of medical grade silicone rubber and fluoroplastic.

Provox Vega package

The Provox Vega package contains the following items:

- 1 Provox Vega voice prosthesis pre-loaded in a single-use SmartInserter, sterile (Figure 2)
- 1 Provox Brush of a size corresponding to the voice prosthesis, non-sterile
- 1 Provox Vega Clinician's manual
- 1 Provox Vega Patient's manual
- 1 Provox Brush Instructions for Use

1.3 CONTRAINDICATIONS

There are no known contraindications for use or replacement of the Provox Vega voice prosthesis among patients already using prosthetic voice rehabilitation.

1.4 WARNINGS

Dislodgement or extrusion of the ProvoxVega voice prosthesis from the TE puncture and subsequent ingestion, aspiration or tissue damage may occur.

To reduce the risk of dislodgment/extrusion and its potential sequelae:

- **Select the proper prosthesis size.** A tight fit may cause tissue necrosis and extrusion.
- **Instruct the patient to consult a physician** immediately if there are any signs of tissue edema and/or inflammation/infection.
- **Instruct the patient to consult a physician** if leakage through or around the voice prosthesis occurs. Leakage may cause aspiration pneumonia.
- If used, **choose laryngectomy tubes or stoma buttons with a suitable shape** that do not exert pressure on the prosthesis or catch onto the tracheal flange of the prosthesis during insertion and removal of the laryngectomy tube or stoma button.
- **Instruct the patient to use only genuine Provox accessories of corresponding size** (Brush, Flush, Plug) for maintenance and to avoid all other kinds of manipulation.
- Re-use and re-processing may cause cross-contamination and damage to the device, which could cause patient harm.

1.5 PRECAUTIONS

- Carefully assess any patient with bleeding disorders or who is undergoing anticoagulant treatment for the risk of bleeding or hemorrhage prior to placement or replacement of the prosthesis.
- Inspect the package before use. If the package is damaged or opened, do not use the product.
- Always use aseptic technique when handling the prosthesis in order to reduce infection risk.
- Make sure that any external or internal stoma attachment devices used do not exert pressure or hook the flange of the prosthesis (e.g., HME base plates, laryngectomy tubes, or stoma buttons). This may lead to severe tissue damage and/or accidental ingestion of the prosthesis.

2. Instructions for use

2.1 Preparation

Choosing the right shaft diameter and length of the replacement prosthesis

Be sure to use a Provox Vega voice prosthesis of the proper shaft diameter and length. Provox Vega is available in different shaft diameters and in several lengths.

- **Shaft diameter**

The Clinician should determine the proper diameter of the prosthesis appropriate for the patient.

- If the selected diameter is larger than the previous prosthesis the tract must be dilated using the Provox Dilator appropriate to the diameter of the prosthesis being inserted.
- If a prosthesis with a smaller shaft diameter is inserted, it is important to observe and ensure that the puncture shrinks to the appropriate diameter.

- **Shaft length**

To select the correct length, you may use the current prosthesis as its own measuring device. If there is too much (i.e. 3 mm / ~0.12 inches, or more) space between the Tracheal Flange of the old prosthesis and the mucosal wall, a shorter prosthesis should be used. If the prosthesis sits too tight, a longer prosthesis should be used.

2.2 Anterograde replacement procedure

(Figures 3a-g)

Activating Provox SmartInserter for replacement

With one hand holding the loading tube, use the other hand to push the inserter pin partially into the loading tube, until the blue ring of the prosthesis has passed the step marker of the loading tube (figure 3a). The Provox Vega voice prosthesis is now ready for insertion.

Preparing the puncture (optional)

The puncture may be dilated to prepare for the insertion of the voice prosthesis. This is usually not necessary but may facilitate insertion in patients with angled or tight punctures that easily collapse.

Inserting the new prosthesis into the puncture

The current (old) prosthesis is removed from the TE-puncture by pulling out the device with a non-toothed hemostat.

1. **Probing the TE-puncture**

Hold the Provox SmartInserter by the Loading Tube. Gently insert the Tip of the Loading Tube (Figure 3b). Proceed with care if you encounter resistance during insertion of the Loading Tube into the puncture. In this case, dilatation and/or lubrication can ease the insertion.

2. **“Injecting” the prosthesis**

With one hand holding the loading tube stable in this position, use the thumb of the other hand to advance the Inserter Pin until it is completely inserted into the SmartInserter (Figure 3c).

3. **Releasing the prosthesis**

Pull the SmartInserter straight out from the puncture (Figure 3d). The prosthesis remains in the puncture; still firmly attached to the Inserter Pin (Figure 3e).

4. **Checking the proper fit**

Check to see if the prosthesis is in the proper position by rotating and exerting slight traction on the Tracheal Flange (Figure 3e). The oval shape of the Tracheal Flange is helpful in determining the ideal position of the prosthesis (by pointing downward, Figure 3g).

5. **Test proper function**

Test proper function of the prosthesis by asking the patient to speak and by observing the prosthesis while the patient drinks water (Figure 3f).

6. **Cut the Safety Strap with scissors**

When the prosthesis is safely in place, use scissors to cut off the Safety Strap on the outer edge of the Tracheal Flange (Figure 3g). The oval end of the Tracheal Flange should point downwards into the trachea. The Provox Vega voice prosthesis is now ready for use.

2.3 Anterograde replacement procedure using voluntary overshooting

(Figures 4a-h)

When the puncture tract is very short, or when mucosa folds over the posterior part of the prosthesis, or when the esophageal flange is difficult to develop within the esophagus, overshooting can help to completely develop the esophageal flange of the prosthesis. Overshooting means that during the insertion procedure, the Provox Vega voice prosthesis is pushed entirely through the puncture into the esophagus. The prosthesis Safety Strap remains thereby attached to the Inserter Pin.

Activating System for replacement using voluntary overshooting

1. With one hand holding the loading tube, use the other hand to push the inserter pin partially into the loading tube, until the blue ring of the prosthesis has passed the step marker of the loading tube (Figure 4a).
2. Squeeze the Guide between thumb and index finger (Figure 4b). Push it further into the loading tube (Figure 4c). The Provox Vega voice prosthesis is now ready for overshoot-insertion.

Inserting the new prosthesis into the puncture - Overshoot

1. **Probing the TE-puncture**

Hold the Provox SmartInserter by the Loading Tube. Gently insert the Tip of the Loading Tube. Proceed with care if you encounter resistance during insertion of the Loading Tube into the puncture (Figure 3b). In this case, dilatation and/or lubrication of the puncture can ease the insertion

2. **“Injecting” the prosthesis**

With one hand holding the loading tube stable in this position, use the other hand to advance the Inserter Pin until it is completely inserted into the SmartInserter (Figure 4d).

As the Guide is being pushed further into the Loading Tube, the Inserter Pin will advance further and release the Provox Vega voice prosthesis completely within the esophagus.

3. **Pulling the prosthesis back**

Pull the Loading Tube out from the puncture (Figure 4e). Pull the Provox Vega voice prosthesis back into the puncture by grasping and pulling the tracheal flange of the prosthesis with a non-toothed hemostat (Figure 4f).

4. **Checking the proper fit**

Check to see if the prosthesis is in the proper position by rotating and applying slight traction on the tracheal flange (Figure 4g). The oval shape of the tracheal flange should be pointed downwards (Figure 4g).

5. **Test proper function**

Test proper function of the prosthesis by asking the patient to speak and by observing the prosthesis while the patient drinks water (Figure 3f).

6. **Cut the Safety Strap with scissors**

When the prosthesis is safely in place, use scissors to cut off the Safety Strap on the outer edge of the tracheal flange (Figure 3g). The oval end of the tracheal flange should point downwards into the trachea (Figure 3f). The Provox Vega voice prosthesis is now ready for use.

CAUTION: If reloading the SmartInserter after Overshooting, make sure the Folding device and Guide are assembled in proper way (see section 2.4 below).

2.4 Reloading the Provox SmartInserter

(Figures 5a-i)

If the first attempt to insert the Provox Vega voice prosthesis into the TE-puncture was unsuccessful, it is possible to reload the Provox Vega into the SmartInserter.

CAUTION: Do not reload the SmartInserter more than 2 times. If the Provox Vega voice prosthesis shows any signs of damage, do not use the prosthesis and obtain a replacement prosthesis.

1. Make sure the prosthesis is still safely attached to the Inserter Pin (Figure 5b1-5b2). If not, secure the Safety Strap on the Inserter Pin by leading the Safety Strap through the Attachment Hole in the Inserter Pin (Figure 5c-d).
2. Put the Provox Vega voice prosthesis with the tracheal side down on top of the Inserter Pin (Figure 5e-f). Use your thumb

to ensure that the voice prosthesis is properly positioned (Figure 5f).

3. Check that the Folding Device and Guide are still assembled in the upper Guide position. If not, re-assemble the Folding Device into Guide according to figure 5g. Do not push the Folding Device in the bottom position of the Guide.
4. Gently pull the Inserter Pin with the attached prosthesis into the Folding Device until you can clearly hear it click in place (Figure 5h). The Esophageal Flange of the prosthesis is now in level with the Folding Device.
5. Attach the Loading Tube to the top of the Folding Device / Guide (Figure 5i) and let the two parts snap together.

The Provox Vega voice prosthesis is now ready for activation and can be used as described in section “**Anterograde Replacement Procedure**” or “**Anterograde Replacement Procedure including voluntary overshooting**”.

3. Important patient information

3.1 General information

Ensure that the patient understands to contact their clinician if:

- There is leakage through or around the prosthesis (coughing and/or change of mucus color).
- Speaking becomes difficult (higher effort and/or voice sounds more strained).
- There are any signs of inflammation or tissue changes at the puncture site or the stoma region (pain, redness, heat, swelling, traces of blood on the brush after brushing).

Also inform the patient that:

- After a prosthesis replacement traces of blood may be found in the sputum. This may come from granulation tissue on the edges of the TE puncture.
- Occasionally, mild leakage through or around the prosthesis may occur in the first weeks after insertion of a new prosthesis. This often resolves spontaneously and does not require immediate replacement of the prosthesis.
- If leakage spontaneously occurs through the prosthesis a temporary blockage can be done with a Provox Vega Plug.
- Speech Training Sessions with a trained clinician are advised in order to acquire optimal voice sound, fluent speech, and optimal intelligibility.

3.2 Prosthesis maintenance

Instruct the patient when and how to clean his or her Provox Vega voice prosthesis. Cleaning the prosthesis at least twice a day can help prolong the device life.

CAUTION: Only use genuine Provox accessories that are intended for use with Provox Vega when cleaning the prosthesis.

- Brushing the prosthesis with the Provox Brush twice a day will help to remove mucus and food remnants from the prosthesis.
- Flushing the prosthesis with the Provox Flush also helps to clean debris and mucus from the prosthesis, which helps increase the life of the device.

Note: The Provox Flush is intended for use only by patients who, as assessed by the clinician who prescribes the device, have demonstrated ability to understand and consistently follow the instructions for use without clinician supervision.

- Some dietary measures, like the daily intake of yogurt or butter milk containing lactobacilli, are considered to be helpful against excessive Candida growth.

For detailed information on how to clean the Provox Vega, please see Instructions for each accessory.

4. Adverse events/Trouble shooting information

For information regarding adverse events and trouble shooting, please visit www.provoxweb.info.

5. Additional information

5.1 Compatibility with MRI, X-ray and radiation therapy

Provox voice prostheses have been tested and found to be compatible, except Provox ActiValve, with Magnetic Resonance Imaging (tested up to 1.5 Tesla), x-ray and radiation therapy (tested up to 70 Gy). The prosthesis can be left within the TE-puncture during the examination/therapy session. Note that the radio-opaque valve seat is visible on x-ray.

5.2 Device lifetime

Depending on individual biological circumstances the device life varies, and it is not possible to predict the integrity of the device over a longer period of time. The material of the device will be affected by e.g. bacteria and yeast, and the structural integrity of the device will eventually deteriorate.

Laboratory testings of simulated usage for a test period of 12 months show that, in the absence of bacteria and yeasts, the device will maintain its structural integrity for this time period. The device has

not been tested for usage beyond 12 months. Usage beyond this limit is under the sole discretion of the prescriber.

5.3 Compatibility with antifungal medicine

Laboratory tests show only minor negative influence on the function of the Provox Vega voice prosthesis and components when using the following antifungal medications: Nystatin, Fluconazole, Mikonazol.

5.4 User assistance information

For additional help or information please see back cover for contact information.

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© Atos Medical AB, 2017.