

PROVOX[®] Brush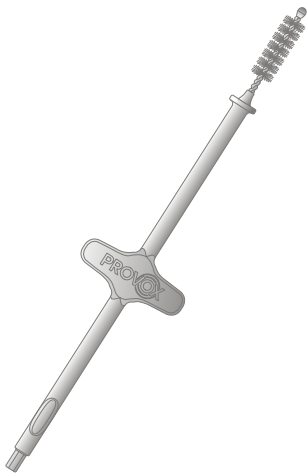





Atos





Atos Medical Your voice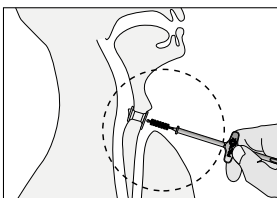

1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28
29	30					

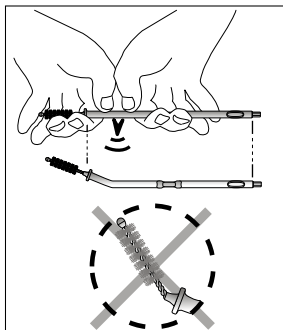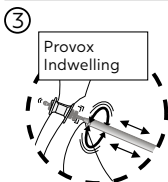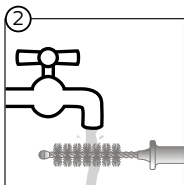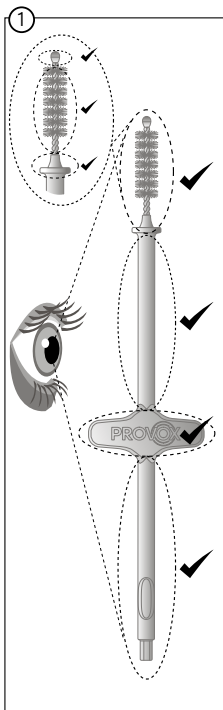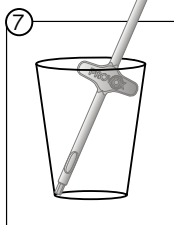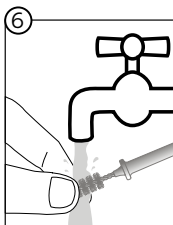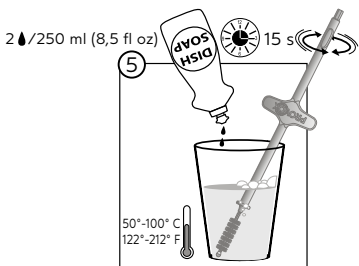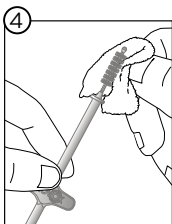8

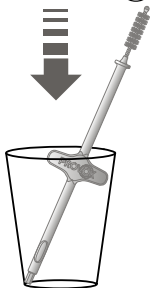
70 % Ethanol  10 min

or

70 % Isopropylalcohol  10 min

or

3 % Hydrogenperoxide  60 min

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox[®] is a registered trademark owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents

Contents

ENGLISH	8
DEUTSCH	9
NEDERLANDS.....	10
FRANÇAIS	11
ITALIANO.....	12
ESPAÑOL	13
PORTUGUÊS.....	14
SVENSKA	15
DANSK.....	16
NORSK.....	17
SUOMI	18
EESTI	19
LIETUVIŲ K.	20
ČESKY	21
MAGYAR	22
POLSKI.....	23
HRVATSKI	24
ΕΛΛΗΝΙΚΑ	25
TÜRKÇE	26
РУССКИЙ	27
日本語	28
한국어	29
繁體中文	30
简体中文	31
עברית	32
العربية	33
AZƏRBAYCAN DİLİ	34
ՀԱՅԵՐԵՆ	35
BAHASA INDONESIA	36
ÍSLENSKA	37
ქართული.....	38
ROMÂNĂ	39
SLOVENČINA	40

Before use

Always inspect the Provox Brush (fig. 1) to make sure that it is not broken or worn out.

Before each use, make sure that

- the protective tip is not cracked or loose.
- the bristles do not appear worn or are loose.
- the wires are not bent or broken.
- the entire handle is not cracked or broken and that the safety wings have not been removed.

If any defects are found, the brush must be discarded.

Cleaning and disinfection

Clean the Provox Brush after each use according to fig. 4-7 using drinking water.

Disinfect the Provox Brush (fig. 8) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Do not use the device until it is completely dry

Note: Always clean and disinfect before use.

Precaution

- The device is intended for single patient use only. Reuse in another patient may cause cross contamination.
- Do not clean or disinfect by any other method as this might cause product damage and injury.

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© Atos Medical AB, Sweden.