




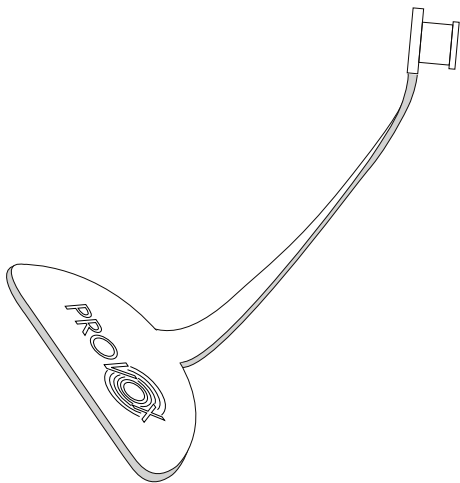
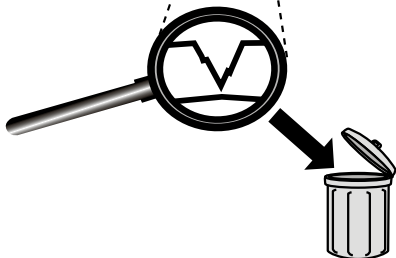
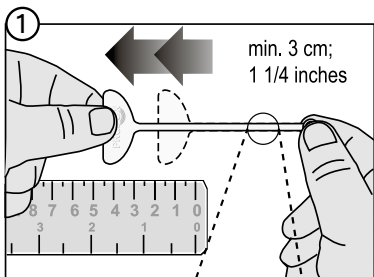
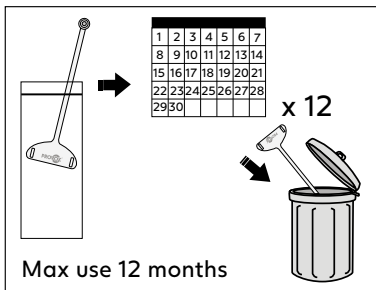
# PROVOX<sup>®</sup> Plug






# Atos

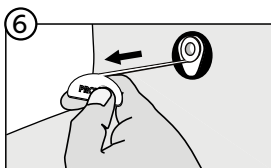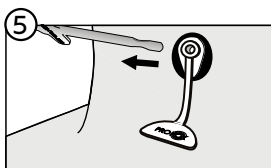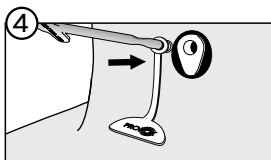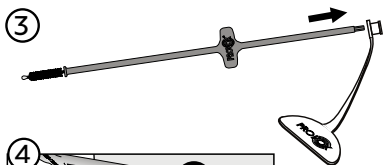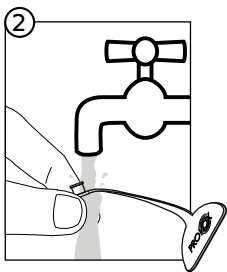Atos Medical **Your voice**

2 / 250 ml (8,5 fl oz)
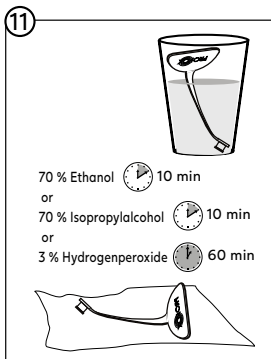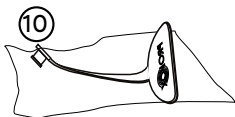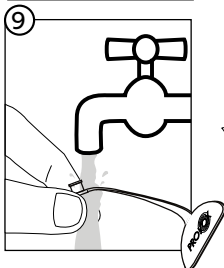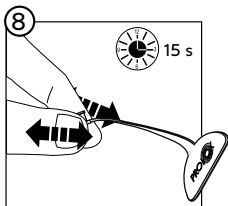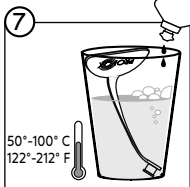
## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and trademarks**

Provox<sup>®</sup> is a registered trademark owned by Atos Medical AB, Sweden.

For information about protective rights (e.g. patents), please refer to our web page [www.atosmedical.com/patents](http://www.atosmedical.com/patents).

# Contents

ENGLISH .....	7
DEUTSCH .....	9
NEDERLANDS.....	11
FRANÇAIS .....	13
ITALIANO.....	15
ESPAÑOL .....	17
PORTUGUÊS.....	19
SVENSKA .....	21
POLSKI.....	23
РУССКИЙ .....	25
日本語 .....	27
<b>한국어</b> .....	29
繁體中文 .....	31
עברית.....	34
العربية.....	36

The Provox Plug is intended for use with Provox Voice Prostheses, Provox2 Voice Prostheses and Provox ActiValve.

Check the device integrity prior to each use by pulling the strap (fig. 1).

## **Precaution**

- The device is intended for single patient use only. Reuse in another patient may cause cross contamination.

## **Cleaning and disinfection**

Clean the Provox Plug after each use, or if it is removed during use according to fig. 7-10.

Do not use any water other than drinking water to clean and rinse the device.

Disinfect the Provox Plug at least once a day if it is removed (fig. 11) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

**Caution:** Do not clean or disinfect by any other method as this might cause product damage and injury.

If the Provox Plug looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use. A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect the Provox Plug after use but also **before** use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

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© Atos Medical AB, Sweden.