

DRINK-RITE®

...the safer drinking system



Drink-Rite® Cup option

Drink-Rite® Beaker option



Providing safe, controlled flow of fluids

- The Drink-Rite System provides the safety and knowledge that users can start to take fluid carefully and without the risk of tilting the head.
- This is important particularly after stroke and in the care of the elderly.
- Children with developmental issues will also benefit from the design features of Drink-Rite.
- Drink-Rite provides a controlled limited flow of fluid into the mouth with a choice of 5cc and 10cc every time you tilt the cup/beaker.
- Ideal for domestic as well as nursing home use.
- Available in cup and beaker options.

www.kapitex.com



KAPITEX

Adding Quality of Life to
Airway Management



DRINK-RITE®

Delivery and a fixed amount of fluid every time

Why Drink-Rite is important

- Drink-Rite provides a controlled flow of fluid into the mouth. This is important, as some users will not be able to drink and swallow safely if there is too much fluid delivered. This often occurs after stroke or other head and neck issues.
- The epiglottis helps protect the airway whilst swallowing and it may not respond as quickly after a stroke. Too much fluid in the mouth in the early stages of stroke recovery could be a problem as fluid may enter the airway (lungs).
- With small amounts of fluid being delivered every time the Drink-Rite cup/beaker is tilted, it allows the user to take fluid carefully, reducing the risks of coughing and choking, thereby relieving the stress and discomfort when drinking.

Stage 1

- Initially - the 5cc insert would be used. This is an effective and safe start - to encourage hydration. With the use of the 5cc insert - small but regular intakes of fluid can take place.

Stage 2

- After the user has made progress in their rehabilitation a larger amount of fluid could be taken - this is facilitated by the 10cc insert.

Drink-Rite delivers the following benefits:

- ✓ Specially designed to encourage safe delivery of fluids
- ✓ Controlled flow, 5cc or 10cc every time
- ✓ Clear cup/beaker - so you can see what is delivered
- ✓ Wide handles to allow easy hold and grip
- ✓ Smooth, no sharp edges, shaped for the lips
- ✓ Spout shaped to encourage lip seal
- ✓ Dishwasher proof
- ✓ Bite-safe and shatter proof

Not suitable for microwave use

Available in cup and beaker options



5cc cup (blue) delivery insert



5cc beaker (blue) delivery insert



10cc cup (green) delivery insert



10cc beaker (green) delivery insert

Drink-Rite® Cup
 Catalogue No. OS FSA 5301
 Comprising of cup, 5cc insert,
 10cc insert

Drink-Rite® Beaker
 Catalogue No. OS FSA 5302
 Comprising of beaker, 5cc insert,
 10cc insert

(Patent Applied for)

Dysphagia (swallowing difficulties) is a serious medical issue, advice should be sought from healthcare professionals.



Kapitex have a wide range of medical devices aimed at reducing the difficulties of swallowing, and have systems aimed at Oral Motor rehabilitation as well as chewing skills.



KAPITEX®

Adding Quality of Life to
 Airway Management

www.kapitex.com

Kapitex Healthcare Ltd. Kapitex House, 1 Sandbeck Way, Wetherby, West Yorkshire LS22 7GH, UK
 Tel: +44 (0)1937 580211 Fax: +44 (0)1937 580796 • Email: sales@kapitex.com